


ET-CARE™ Endotracheal Tube FIXATION Device
APPLICATION INSTRUCTIONS
“SAVE THE FACE”

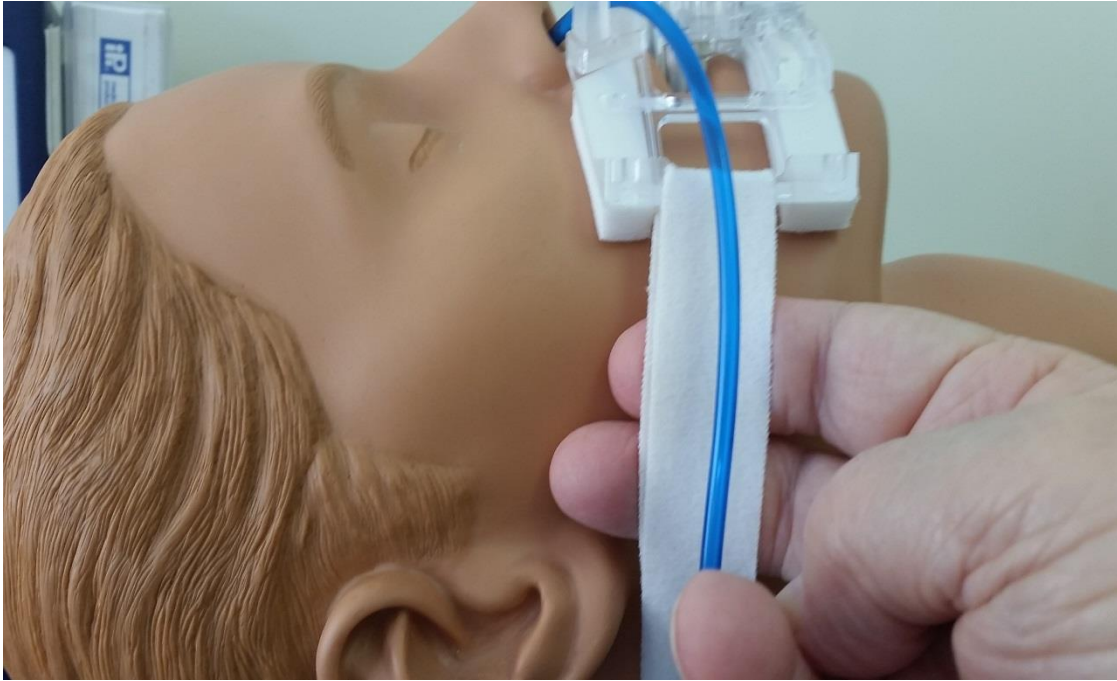

DO NOT OVER TIGHTEN—DO THE TWO FINGER TEST

Proper head gear tension is critical in reducing skin issues that can be associated with invasive or non-invasive ventilation

Note: See additional instruction sheet enclosed with product

PATIENT SKIN ASSESSMENT TO PERFORM PROPER PROTOCOL



-Center face plate on patient. Pass pre-assembled (right side) neck strap behind the neck and secure at left side of face plate post and attach neck strap to the Velcro tab **(see red dot)** under left ear

-Secure the ETT to face plate by placing the ETT retaining straps over the ETT

-Perform two finger test to insure proper headgear tension

IPI MEDICAL PRODUCTS, div. Inhalation Plastics, Inc.

Phone: 561-330-7820 • www.ipimedicalproducts.com • info@ipimedicalproducts.com

MADE IN THE USA (Patent Rights Reserved)